

A proven healthy pleasure for a more intense orgasm

## Instructions for use of Whoop-de-doo Venus Balls



Use of the **Whoop-de-doo** Venus Balls strengthens the pelvic floor and vaginal muscles, which has a positive influence on sexual enjoyment.

The movements you make when using the Venus Balls increase the effectiveness of the strengthening. With intense movements, the user can experience intense arousal and even orgasm. The Venus Balls also help to prevent incontinence.

The Venus Balls are the ideal solution for women who don't have the time or patience for regular exercising of the pelvic floor for a sufficient amount of time. Simply insert the Venus Balls into the vagina; the rest happens on its own without the need for active participation.

The **Whoop-de-doo** Venus Balls work on the time-proven principle of a ball within a ball. With each movement, the internal ball creates stimulating vibrations **A** that pleasantly massage the vaginal walls and improve circulation. The massaging action can be easily altered through spontaneous or regular muscle contractions, both of which strengthen the pelvic floor.

The **Whoop-de-doo** Venus Balls can be used while walking, dancing,

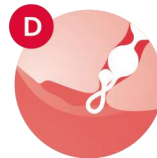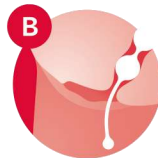

practicing yoga, etc. If you want to achieve arousal while using the Venus Balls, you can stimulate your clitoris with the silicone string.

The **Whoop-de-doo** Venus Balls are made from hypoallergenic medical grade silicone. Unlike gel or latex, medical grade silicone does not allow bacteria to adhere to it and it adjusts to body temperature very quickly.

Before using the **Whoop-de-doo** Venus Balls, wash them thoroughly with

lukewarm water. To use the Venus Balls for sexual arousal or for easier insertion, we recommend using **Whoop-de-doo** Lubricating Gel or another water-based lubricant. Other types of lubricant may damage the surface of the balls.

First, insert the ball that does not have the silicone string on it. Then insert the balls to a depth that is comfortable for you, but only so deep that you can easily remove them at any time using the attached string **B**. The soft material means that the string can be positioned against the clitoris **C** or coiled **D** when using the balls for an extended period of time. The balls can then be removed by simply forming a loop with the string and gently pulling them out.

We recommend using the **Whoop-de-doo** Venus Balls 2-3x per week for 2-3 hours at a time. Wash them thoroughly after every use. To maintain the balls, use antibacterial soap and clean them under running water. If necessary, **Whoop-de-doo** Venus Balls can be disinfected in boiling water for 3-5 minutes.

Store them in their soft, breathable cotton shell (swiss+cotton). The balls should be stored in a cool, dark, dry place. Do not use the balls if their surface has been damaged or torn. The **Whoop-de-doo** Venus Balls are not a children's toy and should be kept out of reach of children.



*The **Whoop-de-doo** Venus Balls were designed and are manufactured from non-toxic materials in the Czech Republic. They were tested at the Institute of Public Health in Prague.*

*Development of the product was carried out in consultation with gynecologists.*

### Package contents

*Venus Balls, protective cotton shell, instructions for use, certificate of authenticity.*

 whoopeddoo love

 facebook.com/whoopeddoo cz

 instagram.com/whoopeddoo cz